

New Segment CMR release



We are excited to announce a new version of Segment CMR!

This new version includes new options in AI AutoMate, improvements in patient reporting, added option to only show relevant stacks, and many improvements in the analysis workflow.

Thank you for helping us create a better software through your feedback!


Important IT information

Port used for communication with license server changed from port 80 to port 443.

If the software is not able to reach the license server after upgrade, then ensure that port 443 is open to the URL <https://medviso.com>

Updated Features


AI AutoMate

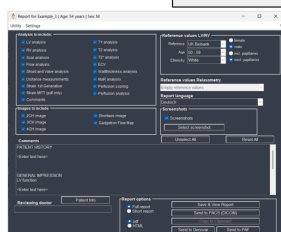

Only show SAX



Copy to all

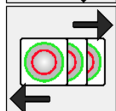

- New options in AI AutoMate
 - New feature to import with AI AutoMate to Segment database
 - Default settings found under Preferences menu
- Improvements in patient reporting
 - Added option for more reference values
 - Added information about incl. or excl. of papillary
 - Added BMI value
 - Added default settings for reporting in preferences
- Option to only show relevant stacks. Feature found by new selection of ALL / MODE in the image stack thumbnail panel. Hot-key "Ctrl-space" to toggle between ALL and MODE
- Improvements in analysis workflow
 - New icon to hide all except SAX cine segmentation
 - In T1/T2/T2* tab, added icon to copy segmentation from one time frame to all time frames
 - Improved tools for adjusting speed of movie in main interface, features found in lower panel in main interface
 - Display of up to 35 ROI info in montage view
 - Hot-key "Shift-space" to toggle between LV/RV/LA/RA
 - Hot-key "Ctrl-mouse wheel" for zooming
 - Improved length measurement labelling and visualisation and possibility to move the label

We are confident that these updates will greatly improve your workflow and analytics, resulting in even more robust patient reports. Thank you for choosing Segment CMR as your preferred software solution for quantitative cardiac MR analysis.

Software Upgrading

- **Note** After upgrading, you must start Segment CMR by "Run as administrator" the first time to finalise the installation
- System requirements for Segment CMR v.4.2 are found in [Segment CMR manual](#) Chapter 3.1.
- Log in to our [download homepage](#) to download and run the installation file.
- Contact us at [support](#) if you would like to have assistance in the upgrading process.

Medviso Newsletter

You are receiving this newsletter since you are registered as a user of Medviso's software packages. If you do not want to receive further newsletters, please use the link below to unsubscribe.

Best regards

Helen Fransson, CEO Medviso AB

helen.fransson@medviso.com