
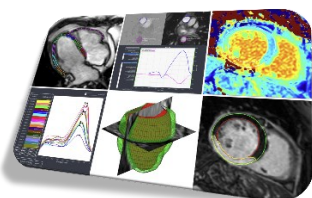


New Segment CMR release

We are excited to announce a new version of Segment CMR! This new version includes improvements in relaxometry analysis workflow, and more choices and results to the patient report.

Thank you for helping us create a better software through your feedback!

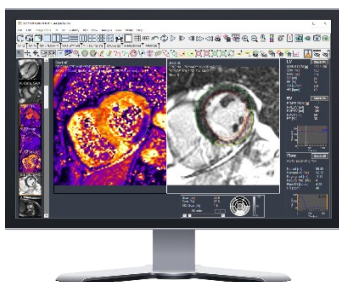

- Improvements in relaxometry analysis workflow
 - Reduced user input for T1/T2/T2* analysis
 - Clarified slice/stack numbering for ECV analysis
- Updates in patient report
 - Added result for maximal flow velocity
 - Provided an option to generate report in Polish
 - Added feature to send report to PACS as a DICOM-encapsulated PDF
 - Relaxometry results are now available in a short report and copy-to-clipboard function
 - Added an option to include reference values for T1/T2/T2*/ECV
 - Provided an option to choose screenshots to be included in a full report
- More stable patient database
- Automatic color assignment for static tissue ROI to orange
- Searching the label-lists by typing the first letter. Useful for ROI-label list.

We are confident that these updates will greatly improve your workflow and analytics, resulting in even more robust patient reports. Thank you for choosing Segment CMR as your preferred software solution for quantitative cardiac MR analysis.

Software Upgrading

- **Note** After upgrading, you must start Segment CMR by "Run as administrator" the first time to finalise the installation
- System requirements for Segment CMR v.4.0 are found in [Segment CMR manual](#) Chapter 3.1.
- Log in to our [download homepage](#) to download and run the installation file.
- Contact us at [support](#) if you would like to have assistance in the upgrading process.

Medviso Newsletter

You are receiving this newsletter since you are registered as a user of Medviso's software packages. If you do not want to receive further newsletters, please use the link below to unsubscribe.

Best regards

Karolina Gottschalk, Application Specialist Medviso AB
karolina.gottschalk@medviso.com