

New Segment CMR 4.0 release


We are excited to announce the release of the new version of Segment CMR 4.0!

This new version includes many improvements that enhance your daily workflow for image analysis process. Thank you for helping us create a better software through your feedback!



New features and software updates

- Improvements in Patient Report functionality providing better visualization and understanding of analysis results
 - Color-scale bar for relaxometry maps is now presented in the Patient Report
 - LV/RV results are now presented line by line and are highlighted in red if they are outside the reference values in the "Copy to clipboard" function.
- Simplified workflow for generation of ECV maps, now you can generate map with only ROI for blood and LV delineations.
- Improved usability for the interface Select DICOM and Load to provide better guidance for data selection.
- Added a preference to change batch size for AI to adapt computation speed for computer with low and high computational power.
- Improved view storage tool, now auto-zoom and contrast/brightness settings are saved in the storage view.
- Automatic detection of LV rotation is now added as a default suggestion for bullseye plot analysis
- Added "Close multiple stacks" function to the File menu; now you can close several stacks at the same time.

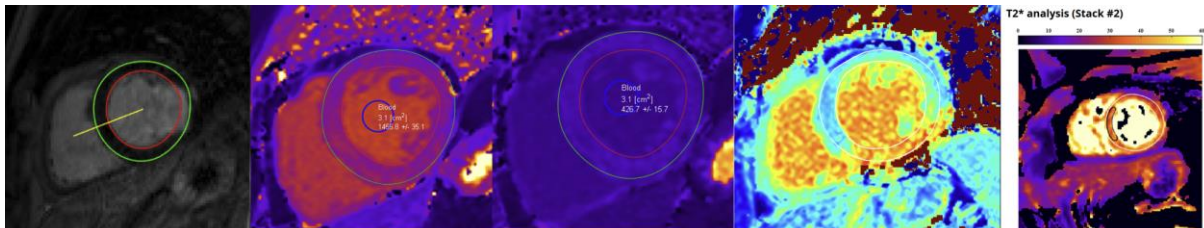

We are confident that these updates will greatly improve your workflow and analytics, resulting in even more robust patient reports. Thank you for choosing Segment CMR as your preferred software solution for quantitative cardiac MR analysis. We value your feedback and suggestions, and we are excited to see how these new features and updates will enhance the work you do.

Software Upgrading

- **Note** After upgrading, you must start Segment CMR by "Run as administrator" the first time to finalise the installation
- System requirements for Segment CMR v.4.0 are found in [Segment CMR manual](#) Chapter 3.1.
- Log in to our [download homepage](#) to download and run the installation file.
- Contact us at [support](#) if you would like to have assistance in the upgrading process.

Medviso Newsletter

You are receiving this newsletter since you are registered as a user of Medviso's software packages. If you do not want to receive further newsletters, please use the link below to unsubscribe.

Best regards

Karolina Gottschalk, Application Specialist Medviso AB
karolina.gottschalk@medviso.com